

## Set up for Windows

1. Follow the [setup instructions](#) for installation of software for NeuroSky MindWave Mobile. You can access the latest info and installation [les here](#)
2. Register for Focus Pocus Account and activate your User Profile



### After the above steps, to run Focus Pocus:

1. Pair the MindWave Mobile with your PC via Bluetooth\*
  - Via Control Panel, Hardware and Sound, Device and Printers, Add a device
  - Blue light on MindWave will be solid if you are paired and connected
2. Put the Mindwave Mobile on your (or your child's) head
  - Ensure forehead sensor is against skin, and give earclip a gentle squeeze
3. Run Focus Pocus
  - If it's the first time you've run the software, you will need to create a Focus Pocus account and establish a User Profile using a Serial Number
  - If you have a Focus Pocus account, login to your account and/or Profile
  - The software will make a connection with the MindWave device

[More detailed instructions are available here](#)

\* If you do not have Bluetooth built-in to your computer, you will need to [obtain a USB Bluetooth Adapter](#)

# FOCUS POCUS

[HOME](#)

[THE GAME](#)

[RESEARCH](#)

[GETTING STARTED](#)

[DOWNLOADS](#)

[FAQS](#)